

# Quick Guide

## Recording images

automatic mode, minimizes



## Recording image

### Recording single image *in automatic mode*




*When the program starts, it is in the background (minimized).*

#### Recording image:



1. Trigger on camera *(Activate/disable in Settings)*
2. USB-Switch *(Activate/disable in Settings)*

#### Modification of camera parameters *(light conditions)*

The parameters can be changed only when the application window is maximized.

-  1. Settings eye complete
-  2. Settings slit lighting
-  3. Settings fluorescence microscopy

#### Automatic adjustment

-  1. Auto contrast on/off *(Activate/disable in Settings)*
-  2. Automatic white balance on/off *(Activate/disable in Settings)*

#### Basic Settings:

camera started  
Application window is minimized  
Camera image original 1:1  
Parameter Full screen