

Quick Guide Recording images and videos



normal mode

Recording image

Recording single image



Start camera



Change **image** directory

Recording image:



1. Save single image
2. Space bar (*Activate/disable in Settings*)
3. Trigger on camera (*Activate/disable in Settings*)
4. USB-Switch (*Activate/disable in Settings*)

Modification of camera parameters (*light conditions*)



1. Settings eye complete



2. Settings slit lighting



3. Settings fluorescence microscopy

Automatic adjustment



1. Auto contrast on/off (*Activate/disable in Settings*)



2. Automatic white balance on/off (*Activate/disable in Settings*)

Recording video



Start camera

*! The **video** directory can only be set via the settings!*

Recording video:



Video start



Video stop

Modification of camera parameters (*light conditions*)



1. Settings eye complete



2. Settings slit lighting



3. Settings fluorescence microscopy

Automatic adjustment



1. Auto contrast on/off (*Activate/disable in Settings*)



2. Automatic white balance on/off (*Activate/disable in Settings*)

Basic Settings

Camera not been started

Normal application window

Camera image original 1:1

Parameter Full screen

Recording single image

Recording video